



## Information For User

Doc.No.:FN-FS010101-IFU Rev.01

### **CAREFULLY READ THESE USER INSTRUCTIONS BEFORE USING THIS FACE SHIELD**

**Applicable to products:**Face Shield

**Product code:**FN-FS01010106

**Manufacturer:** Finess Healthcare Group Co.,Ltd

**Address:**No.1503 A1 Block,No.203 Lantian Road 315000 Ningbo China

**Notified body:** Module B and C2 - SATRA Technology Europe Ltd, Bracetown Business Park, Clonee, D15YN2P. Republic of Ireland (Notified Body 2777).

**Declaration of conformity:** [www.finess-healthcare.com](http://www.finess-healthcare.com).

These products are classed as Category III Personal Protective Equipment (PPE) by the European PPE Regulation (EU) 2016/425 and have been shown to comply with this Regulation through limited testing against the Harmonised European Standard EN 166:2001 (see below).

This product is designed to provide some protection against contact with airborne droplets that may constitute a biological hazard. **However, always remember that no item of PPE can provide full protection and care must always be taken while carrying out the risk-related activity. Never use this visor for protection against other hazards for which the product is not design for or tested against**

**PERFORMANCE AND LIMITATIONS OF USE:** These products have been tested in accordance with EN 166: 2001 Clauses: 6 | Design; 7.1.1 | Field of vision; 7.1.2.1 | Refractive powers; 7.1.2.2 | Transmittance; 7.1.2.3 | Diffusion of light; 7.1.3 | Quality of materials; 7.1.5.1 | Stability at elevated temperature; 7.1.7 | Resistance to ignition;7.2.4 | Droplets and splashes of liquids,7.3.2 | Resistance to fogging of oculars

**FITTING AND SIZING :** To put on and take off products. Only wear products of a suitable size that are correctly adjusted. Products which are too loose may not stay in place and if too tight will be uncomfortable. The size of these products is marked on the label.

**COMPATIBILITY :** To optimise protection, it will be necessary to use these products with suitable gloves/gowns/masks. In this case, before carrying out the risk-related activity, consult your supplier or supervisor to ensure that all your protective products are compatible and suitable for your application. Warning - Parts of the visor that come into contact with your skin may lead to allergic reactions. In this case please discontinue use and seek medical advice.

**STORAGE AND TRANSPORT :** When not in use, store the visor in a well-ventilated area away from extremes of temperature. Never place heavy items on top of it. If possible, avoid excessive folding and preferably store it hanging vertically. If the product is wet, allow it to dry fully before placing it into storage.Store the product in a dark place,away from sunlight,when not in use.

**OBSOLESCENCE :** This visor has a shelf life of 3 years. Service life will depend on usage conditions but the visor should be disposed of and replaced if badly scratched or damaged in some way that prevents safe use

**REPAIR:** If the visor becomes damaged, it will NOT provide the optimum level of protection, and therefore should be immediately replaced with a new one.Never use the damaged product.

**CLEANING :** It is single use and cannot be cleaned

The user information will be supplied with product.